## Thealoz®

This product is a medical device.  
Please read the instructions before use.  
Keep this leaflet. You may need to read it again.

### INDICATIONS

THEALOZ is indicated to hydrate and protect the surface of your eyes, for treatment of dry eye syndrome. For ocular use only.

### SUITABLE FOR

- Adults.
- Children: Parents or carers should administer the product.
- Pregnant or breastfeeding women based on current knowledge.
- Wearers of any type of contact lenses.

### CONTRAINDICATIONS

Do not inject and do not swallow. Do not use THEALOZ if you are allergic to any of its ingredients (see Device description).

### DEVICE DESCRIPTION

THEALOZ is a solution containing trehalose (3%), sodium chloride, trometamol, hydrochloric acid and water for injections. THEALOZ is a sterile, phosphate-free, isotonic and pH neutral solution.

The solution is supplied in the patented ABAX® multi-dose bottle. This innovative design delivers eye drops through a 0.2 µm filter, preventing any bacterial contamination of the solution and avoiding the use of any chemical preservatives. This means THEALOZ is gentle for eyes because it is preservative-free. Avoiding chemical preservatives is beneficial for the eye's surface long term and eliminates reactions amongst people who are sensitive to preservatives.

**BENEFIT, PERFORMANCE AND MECHANISM OF ACTION**  
THEALOZ provides immediate comfort that lasts all day long, to relieve the symptoms of dry eyes.  
THEALOZ is recommended if you have discomfort, stinging, foreign body sensations or irritation of your eyes. These symptoms can be caused factors such as wind, smoke, pollution, dust, sunny or cold conditions, dry heat, air conditioning, plane journeys, and also by long hours working in front of a computer screen or digital device.

THEALOZ is compatible with the delicate surface of your eyes, and can also be used by people wearing any type of contact lenses.

The main ingredient trehalose is a natural substance found in many plants and animals, which need to survive in extreme conditions of temperature and dryness. It provides protective and hydrating properties.

**GENERAL RECOMMENDATIONS FOR USE**  
Apply 1 drop to each eye, as often as necessary during the day.

THEALOZ is easy to use. Wash hands thoroughly before use. Do not use any tool to open the bottle. When you first use THEALOZ, you will find the first drop takes longer to come out due to the unique filtration system in the bottle (see Device description).

Apply one drop in the pocket of the eyelid, while gently pulling the lower lid down and looking up. Close the eye gently after application.

Replace the bottle cap after use.

**CAUTIONS FOR USE**  
Multi-dose bottle, for use by one person only.  
Do not share bottles between different people to prevent cross contamination. Check the packaging is intact before first using this product.

Check the tamper-proof ring on the bottle is intact before using it for the first time. To avoid contamination, do not touch the tip of the bottle to any surface.

You must contact a healthcare professional if your symptoms get worse or do not improve.

**WARNINGS**  
Keep out of the sight and reach of children. After use, do not drive or use machines if your vision is blurred: wait for a few minutes until your vision clears.

Wait at least 10 minutes between using two different eye products. Never dilute THEALOZ or mix it with other solutions.

Once opened, if you expose the bottle to low atmospheric pressure (such as on an airplane), the bottle may leak.

**SIDE EFFECTS**  
The following side effects have been reported: Rare possibility of mild eye irritation and ocular redness. In such cases, contact lens wearers should remove their contact lenses.

When you use eye drops, you might experience disturbing symptoms such as burning sensation, stinging sensation, foreign body sensation in the eye and blurred vision for a short time.

If you notice any side effects or unusual sensations after using this product, please contact a healthcare professional and report the information to the local distributor, to the manufacturer or to your local Health Authority (see contacts at the end of the instructions for use).

**STORAGE AND DISPOSAL CONDITIONS**  
Store between 8 °C and 25 °C. Do not expose your product to direct sunlight or humidity. Do not use after the expiry date, which is marked on the bottle and on the outer box. The expiry date refers to the last day of that month as long as the packaging is intact and it has been stored correctly.

The product can be used until 8 weeks after first opening. After this time, discard any leftover product responsibly.

Dispose of the product responsibly and in accordance with local guidance (the bottle should be disposed of as household waste, the outer carton and the leaflet are recyclable).

**Date of issue and version of the instructions for use: Ver.5 11/2019**

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**Health authority:**  
Health Products Regulatory Authority,  
Kevin O'Malley House, Earlsfort Centre,  
Earlsfort Terrace, Dublin 2, D02 XPT7  
Tel.: 1 676 497

**CE 0459**

## MEANING OF THE PICTOGRAMS ON THE PACKAGING

	Manufacturer
	Medical Device
	Batch number
	Consult instruction for use
	Date of manufacture
	Single patient - Multiple use
	Use-by date
	Do not use if package is damaged
	Temperature limits
	Caution
	Sterilized using aseptic processing techniques

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Thea Portugal, S.A.